DESCRIZIONE

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EN

DESCRIPTION

Drentech™ Surgical is a single-use surgical drainage system for collection and recovery of autologous blood for immediate autotransfusion.

The system is composed of the following parts:

1. A collection chamber (1) with a maximum capacity of 1.500 ml. The collection chamber has an infusion point on the inlet of the drainage (3) and is fitted with a 120 µm macroaggregate filter (10).
2. A housing for the Autotransfusion Unit (here in after called "bag"), made directly into the system body.
3. The reinfusion bag (2) is composed of the bag and of a rigid

- container which protects it. Inside the bag, there is an integrated 40 µm microembolus filter which is always available during the reinfusion.
4. A reinfusion set, one of those normally used within the hospital, is housed in a special compartment in the body of the drainage system. Inside this special compartment there is also a connection tube between the system for collection and the bag.

INSTRUCTIONS FOR USE

The system is suitable for evacuation of fluids and blood which collect in the natural and/or newly formed cavities following surgery, in order to prevent accumulation. The system is mainly intended for collection and recovery of autologous blood for immediate postoperative autotransfusion. The system can operate by suction, through connection to the Drentech™ Vacuum Unit (afterwards called "Vacuum Unit"), or by gravity. The blood recovered in the collection chamber can be transferred to the reinfusion bag and infused to the patient. The system is fitted with a protection device which prevents recovering the buffy coat together with the blood and hence its subsequent reinfusion to the patient. The system carries out a double filtration of collected blood before the reinfusion. For this reason, the system is fitted with two filters, a macroaggregate and a 40 µm microaggregate filter, which are completely integrated in the system for recovery.

Note: This device can be used in safe conditions only in the field of applications and with the methods indicated in this operating instruction leaflet for the specific type of product. The manufacturer declines all liability relating to improper use or use different from that indicated. This instruction leaflet must accompany the device during its entire lifetime and must always be at hand for consultation.

GENERAL WARNINGS

Do not use if the package has been opened or damaged. Carefully read this instruction leaflet before using the product. Single-use product. Reuse may lead to alteration of performance and risks of cross-contamination.

WARNINGS/PRECAUTIONS

- The system must as a rule be maintained in vertical position, also during transport. In the event of overturning, the system in any case guarantees normal functioning. The blood collected can be recovered, but it is advisable to hold the system in vertical and stable position for about 30 minutes before transfer into the bag in order to encourage separation of the buffy coat portion which may have mixed with the collected blood with the risk of being reinfused to the patient.
- If used by gravity, must be placed below the level of the patient.
- Avoid loops or squashing the patient drainage tube (7).
- Do not connect the system to the patient with the suction already active.
- The physician and the assistant staff must be aware of the possible implications associated with autologous blood transfusion. If complications are suspected, adopt the necessary clinical measures to monitor the coagulation factors.
- It is warmly recommended to observe the "Universal precautions relating to blood and body fluids" for treatment of any blood or similar products.
- As recovered blood is kept at room temperature, the time between the starting of the collection and the starting of the reinfusion must not be higher than 6 hours. In order to always have the information relating to the start of recovery available, a label has been fixed on the system and the bag, on which the essential data such as the name of the patient, date and time of recovery start can be indicated.
- It is recommended to limit the maximum volume of blood collected and infused and not subjected to any treatment to a maximum of 1.500 ml. In the last resort, the volume of blood to infuse must always be decided by the physician in charge.
- It is unadvisable to infuse haemostatic topical agents such as, for example, Thrombin or Avitene™, since they might start coagulation. Do not reinfuse blood containing Betadine or similar liquids for wound irrigation or antibiotics not authorised for parenteral use. Methylmethacrylate, if infused, may cause circulatory collapse; if solid, it may clog the autologous blood recovery system. Should these substances have been used, thoroughly wash out the wound with normal physiological solution before closing it.
- If necessary, anticoagulant solution can be infused through the infusion point (3) at the drainage inlet. It is recommended to use ACD-A (in a dose of 40-60 ml per unit of blood collected or in an equivalent ratio of 1:7) since it is better metabolised. If using heparin, it is recommended to proceed as indicated in the international guidelines (e.g.: AABB - Committee for Autologous Transfusions: add 30,000 units of heparin to a liter of saline solution and add about 15 ml of this heparinised saline solution to every 100

- ml of blood or maintaining an equivalent ratio of about 1:7). The use of anticoagulant is generally recommended in full knee prosthesis surgery, especially where a tourniquet is used and it is loosened just before starting recovery, typically when the wound has already been sutured completely. The following can be done to prevent the formation of clots:
- Loosen the tourniquet at least 15-20 min. before activating suction and recovery; or
 - Immediately after loosening the tourniquet, evacuate 50-100 cc of blood through the theatre aspirator or another device, then connect the Y-connector; or
 - if wishing to keep the tourniquet tightened until surgery has been completed and/or if wishing to recover all the blood from the wound, infuse about 20 cc of ACD-A in the system before activating suction through the infusion point (3).

Normally, after 15-20 min. from loosening the tourniquet it is no longer necessary to add anticoagulant to the blood. If considered expedient, preferably ACD-A or, alternatively solution heparinised in the ways and doses previously described can be added at any time. The ACD-A anticoagulant solution does not require, in patients with preserved hepatic functionality, specific antagonists if a dose of 150-180 ml is not exceeded. Administration of calcium solutions antagonises the effects of any excessive administration of anticoagulant.

- If while transferring the blood into the bag, some difficulty is noticed in the transfer or if only air is transferred, it means that blood sedimentation has possibly occurred and that, therefore, it has thickened at the base of the collection chamber. In these cases the transfer can be facilitated by pressing the red vacuum indicator (17) for 5-10 seconds. As a result, the blood will immediately flow in the connection tube (6) confirming that the transfer has been successfully restarted.
- For the autologous blood transfusion, use the infusion set included in the system. A different set can be used provided that it has the same characteristics.

CONTRAINDICATIONS

Blood recovery and infusion are contraindicated in the following cases:

- Anomalous hepatic and/or renal functionality;
- Malignant wounds;
- Contamination and/or sepsis;
- Use of fluids unsuitable for reinfusion;
- Presence of amniotic fluid;
- Presence of bile;
- Presence of haemostatic agents;
- Excessive haemolysis;
- Coagulopathies;
- Possible gas embolism, microembolism, fat embolism.

PREPARING THE SYSTEM

- The system is packed in a single wrapping. To use the product, take it out of its wrapping gripping the flaps at the top end.
- The system can function by suction only and exclusively when connected to the portable Vacuum Unit. If wishing to use the system by suction, set up a Vacuum Unit following the procedure described in the relevant user manual.
- Insert the Vacuum Unit in the dedicated connector (9).
- Ensure that the Vacuum Unit is off before connecting it.

POSITIONING THE DRAINAGE

- Before positioning the drainage, carefully select the type, the position and the size required. These parameters depend on the type of operation, the physiology of the patient and the experience of the physician.
- Using an aseptic technique, carefully remove the sterile thoracic drainage from its protective wrapping.
- Position the drainage in the wound using an aseptic procedure. With products fitted with trocar needle, use the latter according to the usual positioning technique, working from the inside to the outside.
- For proper positioning use the depth markings indicated on all the drainages.
- Fix the drainage by means of a stitch or plaster.
- Connect the positioned tube to the selected drainage system following the instructions on the relevant instruction leaflet. In the case of drainages fitted with trocar needle, cut the latter and dispose of it before connecting to the drainage system. If necessary, the connector provided in the package can be used (where provided) to make the connection to the drainage system.

When the device is connected to the drainage tubes, i.e. at the beginning of the recovery phase, the patient data and the date and time of starting drainage must be noted down on the label fixed on the side of the system.

COLLECTION START: REGULATING THE SUCTION LEVEL

- Switch on the Vacuum Unit and adjust the suction pressure level as

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LEGEND (EN)

- 01 Collection chamber
- 02 Blood recovery bag with reinfusion filter
- 03 Self-sealing infusion point
- 04 Red connection-tube clamp
- 05 Collection system connection tube
- 06 Vacuum Unit/bag connection tube
- 07 Patient connection tube
- 08 Pierceable caps
- 09 Vacuum Unit connection
- 10 Macroaggregate filter
- 11 Handle
- 12 Unopenable clamp
- 13 Vacuum gauge
- 14 Reinfusion set
- 15 Graduated scale: total drainage
- 16 Graduated scale: infusible blood
- 17 Removable label

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indicated in the user manual.

- Note:** It is advisable to start suction from the two lower suction values in order not to encourage possible haemorrhage. In the case of limited blood loss the suction values can be increased at the discretion of the physician in charge.
2. Creation of the vacuum in the collection chambers can be checked by observing the vacuum gauge (13), which must be deformed.
 - a. The suction pressure level must always be fixed by a qualified physician.
 - b. Suction must be suspended in the following cases: if wishing to continue drainage by gravity only, or during replacement of the collection system.
 3. The suction pressure can be reduced at any time but only on instructions of a qualified physician. To reduce the suction pressure refer to the Vacuum Unit user manual.

DETERMINING THE QUANTITY OF BLOOD RECOVERED AND/OR FLUID DRAINED

The collection chamber of the system has two graduated scales. The first (15), in a black colour, is situated on the side of the collection chamber and always provides an indication of the total quantity of drainage fluid collected up to the moment of reading. A space is provided next to this scale to make any notes (e.g. time of last reading to determine the hourly loss of the patient). The second scale (16), coloured white, is situated in the center of the transparent window of the collection chamber. The origin of this scale is in correspondence to the 70 ml mark of the first scale, i.e. in correspondence to the average quantity of blood that remains in the collection chamber due to the effect of the device that prevents recovery of the buffy coat. In this way, it can always immediately be seen how much blood, in the case of recovery, will be transferred to the bag and subsequently reinfused to the patient.

Note: Note down the value indicated by the white scale, corresponding to the blood which will be recovered and reinfused, before transferring it to the bag. This value can be written on the removable label (17) fixed on the side of the system so that the data is subsequently available.

RECOVERY OF COLLECTED AUTOLOGOUS BLOOD

During operation by suction, drainage and blood recovery are visible directly in the collection chamber of the system. The blood contained in this chamber is conveyed through the macroaggregate filter and hence is available for transfer to the bag for subsequent infusion. The decision to recover the blood must be made by a qualified physician who can make a qualitative evaluation by taking some blood samples.

1. Check that the connection between the bag and the Vacuum Unit has been carried out properly.
2. Ensure that the clamp (4) is open so that the bag is in communication with the collection chamber.
3. Turn the knob on the Vacuum Unit to the COLLECTION position as specified in the Vacuum Unit user manual.
4. The Vacuum Unit will start a normal continuous operating cycle, during which the blood will be transferred from the collection chamber to the bag. During the whole recovery phase the suction pressure will not be neutralised but will remain around the minimum level: thus drainage will not be interrupted, preventing possible occlusion of the catheters. After a short time interval, which is to be considered normal, the blood will start flowing from the collection chamber to the bag. A flashing light on the Vacuum Unit will indicate that the operation is in progress.
5. Recovery ends when the blood flow through the connection tube (5) starts becoming discontinuous (appearance of air bubbles in the connection tube). At that stage close the clamps (4) to interrupt the operation.

Note: Approximately 60-70 cc of haematic fluid will remain in the collection chamber. This residue is a result of the action of the internal system which prevents recovery of the buffy coat, typically composed of lipidic material (see paragraph "Determining the quantity of blood recovered and/or fluid drained"). The recovery operation can in any case be suspended at any time.

6. When recovery has ended, return the knob on the Vacuum Unit to the SUCTION position.
7. The Vacuum Unit will return to the suction pressure value set before the start of recovery, reaching it after a short period of continuous operation, which is to be considered normal.
 - a. If the system remains in the COLLECTION position for longer than necessary, the vacuum in the collection chamber will gradually be neutralised. As soon as the recovery operation has been concluded, return the system to the SUCTION configuration.
 - b. It is recommended that the operation be constantly monitored by medical or nursing staff for its entire duration.

Note: The recovery operation can be restarted later, before

reinfusion, repeating the operations described above.

8. The Redax bag is supplied completely closed and the presence of residual air is therefore extremely limited and does not present contraindications thanks to the rigid container which protects the flexible bag from possible accidental crushing. If the blood recovery operation is not suspended when the first bubbles appear in the connection tube, the lipidic substance retention device might let sterile air flow into the bag. This causes the bag to inflate and might not allow further expansion and, consequently, a second recovery operation. To evacuate the air from the bag, the following operations can be carried out:
 - a. Check that the clamp (4) on the connection tube is closed and that the tap on the Vacuum Unit is in the SUCTION position.
 - b. Disconnect the connection between the Vacuum Unit and the bag (6).
 - c. Reopen the clamp (4) on the connection tube: at this point the system will tend to suck the residual air from the bag.
 - d. When all the air has been evacuated from the bag, the blood contained in the bag can be seen as it tends to rise towards the drainage system: suspend the operation by closing the clamp (4) on the connection tube.
 - e. Restore the connection between the Vacuum Unit and the bag (6).
 - f. At this point the bag is ready to perform another blood recovery operation or for infusion.

IMMEDIATE INFUSION OF RECOVERED BLOOD

When the blood has been recovered in the bag, as described in the previous point, it can immediately be infused to the patient. For the warnings on infusion of autologous blood, heed the warnings given at the beginning of this instruction leaflet. Inside the bag is included a macroaggregate filter (40 µm). For autologous blood transfusion, use the infusion set included in the system. An equivalent filter may be used provided that it has the same characteristics.

1. Disconnect the connection between the Vacuum Unit and the bag (6).
2. Extract the bag container from its housing and unroll the connection tube (5) wrapped in a blister pack that can be thrown away. The bag container is fitted with a clamp so that it can be fixed to an I.V. pole.
3. Extract the reinfusion set included in the system from its sterile package. To perform reinfusion with this set, follow the procedures described in the following points. Should another equivalent set be used, follow the respective instructions for use.
 - a. Use the set immediately after opening the package and removing the protective caps on the ends.
 - b. After using the set once only, dispose of it adopting suitable precautions and acting in accordance with the provisions of the law in force for biologically hazardous waste.
 - c. If the set included in the system is not used, it must be disposed of together with the drainage system and must not be used for other purposes.
4. Close the roller clamp located under the drip chamber, then remove the protective cap of the spike located on the top part of the filter.
5. Remove the cap from one of the two pierceable connectors on the bag and insert the spike in the cap with a decisive action and rotary movement.
6. Remove the protective cap on the Luer-lock connector located on the opposite end of the infusion set.
7. Fill the set, open slowly the roller clamp and let the blood flow from the bag to the drip chamber to about 2/3 of its capacity. When done, close the roller clamp.
8. Hang the bag on the I.V. pole using the special clamp. Let the air escape from the infusion set by opening the roller clamp. Finally, proceed with injection into the veins and reinfusion of the autologous blood contained in the bag.
9. At the end of the operation, extract the spike and close the connector cap.
10. Replace the bag container in its housing. The connection tube can be rolled up and placed in the special compartment in the body of the drainage system.
11. Reconnect the connection between the vacuum unit and the bag (6).

The recovery and reinfusion operation can be repeated twice since the system remains completely closed, provided that both operations occur within and not later than six hours from the start of recovery. The second operation can be carried out with the same recovery and reinfusion methods described and already carried out in the first case, with the difference that preferably the second pierceable connector and a new infusion set should be used. If necessary (e.g. heavy blood loss) the recovery and reinfusion operation can be repeated several times, provided that within and not later than six hours from the start of recovery, and adopting the necessary precautions to maintain sterility during connection and disconnection of the filter to the bag. It is

recommended to limit the maximum volume of blood collected and infused and not subjected to any treatment to a maximum of 1.500 ml.
TAKING BLOOD SAMPLES FOR LABORATORY EXAMINATIONS
In order to evaluate the blood parameters, i.e. the quality of the blood recovered before its infusion, it is recommended to take a sample directly from the infusion set. This operation can be carried out by means of a syringe and using the self sealing cap on the set tube. The blood sample can be taken directly during the filling operation of the set, transferring a small quantity of blood into a test tube by taking it directly from the terminal connector. To prevent obtaining values, in particular of hematocrit, which do not reflect the mean data (e.g. because of the tendency of the erythrocytes to sediment), it is recommended to shake the bag a few times to homogenise the distribution of the blood components before taking the sample.

USING AS DRAINAGE SYSTEM

When the six hours from the start of recovery have elapsed (beyond which the blood contained in the collection chamber is no longer suitable for reinfusion) system can be used as collection system for wound drainage, both by suction (with vacuum unit) and by gravity. To this end, no additional operation needs to be carried out, but the system can continue to operate as previously, remaining completely closed to the external environment. At this point the bag can be permanently sealed in order to prevent accidental infusion. The system remains available for wound drainage until saturation of the collection capacity. To permanently seal the bag and thus avoid the risk of accidental infusion, use the unopenable clamp (12) provided with the system. To tighten, insert the clamp in the tube using the special housings, then close the two ends. From this moment on it will no longer be possible to perform any recovery operation.

Note: Do not use the unopenable clamp (12) for normal closing of the connection tube: for this operation a normal red clamp (4) is provided. Should the unopenable clamp (12) be used, it will no longer be possible to perform any recovery operation.

USING THE SYSTEM BY GRAVITY

The system can also operate by gravity.

1. During operation by suction the system can be switched to operation by gravity at any time, but always to be decided by a qualified physician.
2. Suspend suction as described in the corresponding paragraph in these instructions for use.
3. Remove the vacuum unit from its housing. Thus the air can escape from the system allowing operation by gravity. A filter will hold the system closed preventing contamination.
4. During operation by gravity, the system must be positioned below the level of the patient.
5. Suction can be restored as follows:
 - a. Reposition the vacuum unit in its housing.
 - b. Restore suction as described previously.

REPLACING THE UNIT

The collection chambers of the system have a maximum capacity of 1.500 ml. When this capacity has been fully used up and drainage has to be continued, the system must be replaced.

1. To carry out this operation, prepare the new unit following the instructions in the paragraph "Preparing the system" so that it is ready for use.
2. Suspend suction and disconnect the Vacuum Unit from the system in use.
3. Close tube of the Y-connector with a normal tube clamp.
4. Disconnect the system in use acting on the lock-type connector and replace it with the new one.
5. Open the tube clamp.
6. Reconnect the Vacuum Unit and restore suction.

REMOVING THE DRAINAGE

1. Remove the tape or stitch or any other type of fixing.
2. Disconnect the drainage system.
3. Remove the drainage by pulling slightly and immediately apply a wound dressing.

STERILE - Sterilized by ethylene oxide.

The product is sterile unless the package has been opened or damaged. Single-use device. Discard after each single use, do not re-use. Avoid exposure to high temperatures and ultraviolet rays during storage. For discarding and disposal of the device, adopt suitable precautions and act in accordance with the provisions of the law in force for biologically hazardous waste.

MATERIALS USED

PS, PP, PEHD, ABS, PVC,
LATEX-FREE MEDICAL DEVICE.

Date of issue of the last version :
See last page : (REV.: XX-XXXX)

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