REDAX®

IT

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DESCRIZIONE

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Date of issue of the last version:
see last page : (REV.: XX-XXXX)

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INSTRUCTIONS FOR USE

DESCRIPTION

The surgical drainage is a sterile and single-use device. It is available in numerous versions which differ between them in the profile of the tube segment and/or in the material of which they are made (silicone or PVC). All the versions are available in different sizes and are supplied in double sterile packaging. Versions are available with the trocar needle already mounted to the drainage in order to facilitate positioning with aseptic technique. All the drainages have depth markings, graduated from the end of the perforated section: these markings facilitate accurate positioning of the drainage in the wound. All the drainages, both silicone and PVC, are biocompatible and hemocompatible (in accordance with the regulations in force). All the subject products are compatible for use with gravity or suction drainage systems.

INDICATIONS

The surgical drainages are used in numerous different fields of application with the purpose of evacuating potentially pathological fluids and blood that has accumulated in natural and/or newly-formed cavities following surgery. All the surgical drainages are compatible for use in conjunction with postoperative blood recovery systems with the purpose of autotransfusion.

Note: This device can only be used in safety conditions in the field of applications and using the procedures indicated on this operating instruction sheet for the specific type of product. The manufacturer declines all liability relating to improper use or use different to that indicated.

SUGGESTIONS FOR USE

Positioning the drainage.

- Before positioning the drainage, carefully select the type, the position and the size required. These parameters depend on the type of operation, the physiology of the patient and the experience of the physician.
- Using an aseptic technique, carefully remove the sterile thoracic drainage from its protective wrapping.
- Position the drainage in the wound using an aseptic procedure. With products fitted with trocar needle, use the latter according to the usual positioning technique, working from the inside to the outside.
- For proper positioning use the depth markings indicated on all the drainages.
- Fix the drainage by means of a stitch or plaster.
- Connect the positioned tube to the selected drainage system following the instructions on the relevant instruction leaflet. In the case of drainages fitted with trocar needle, cut the latter and dispose of it before connecting to the drainage system. If necessary, the connector provided in the package can be used (where provided) to make the connection to the drainage system.

Removing the drainage.

- Remove the tape or stitch or any other type of fixing.
- Disconnect the drainage system.
- Remove the drainage by pulling slightly and immediately apply a wound dressing.

CONTRAINDICATIONS

There are no known contraindications.

GENERAL PRECAUTIONS

- The product is sterile unless the package has been opened or damaged. Do not use if the package has been opened or damaged.
- Exposure to elevated temperatures and ultraviolet light should be avoided during storage.
- For discarding and disposal of the device, adopt suitable precautions and act in accordance with the provisions of the law in force for biologically hazardous waste.
- Single-use product. Reuse may lead to alteration of performance and risks of cross-contamination. Dispose of the device immediately after use. Do not reuse.

STERILE - Ethylene oxide gas sterilized.

MATERIALS USED

PVC and silicone biocompatible and hemocompatible.

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